



Fresenius Medical Care SERVICE BULLETIN

Copyright © 2005 Fresenius USA, Inc

Equipment: GranuFlo
Bulletin: 05-GRN-003 Rev. A
Subject: Specific Gravity Range for New Concentrate

Originated By (Dept):	Reviewed By:	Approved By:	Effective Date:
J. Fox (TSD)	Brooks E. Rogers	ECO# 16937	9/12/05

Distribution: Dir TS (B. Rogers), TS Mgr (K. Cogswell), QS (M. Owen), QE (R. Fox), Eng (D. Gavin), Prod MGR (R. Bryant), Tng (B. Hassard)

1.0 PURPOSE

To inform all Field Service and Technical Support personnel of the specific gravity range of the new GranuFlo Dry Acid Concentrate 0FD1067-3B.

2.0 SCOPE

A new GranuFlo Dry Acid Concentrate part number 0FD1067-3B is available. The specific gravity range for this concentrate is:

1000 SERIES 1:35.83 Proportioning		
Specific Gravity Range		
Part #	Low	High
0FD1067-3B (NP)	1.148	1.156

On page 2 of this bulletin is the table of limits for all formulations including the new 0FD1067-3B concentrate.

Table 1 on page 2 will be incorporated into the operator's manuals for both the 99-gallon and 132-gallon dissolution systems.

If there are any questions regarding this bulletin, contact Fresenius Technical Support at 800-227-2572 x7003.

Specific Gravity Ranges

All values are at 20°C (68°F)

1000 SERIES 1:35.83 Proportioning

Specific Gravity Range

Part #	Low	High
0FD1003-3B (NP)	1.148	1.156
0FD1008-3B (NP)	1.148	1.156
0FD1018-3B (NP)	1.133	1.141
0FD1029-3B (NP)	1.148	1.156
0FD1041-3B (NP)	1.147	1.155
0FD1042-3B (NP)	1.147	1.155
0FD1052-3B (NP)	1.146	1.154
0FD1062-3B (NP)	1.149	1.157
0FD1067-3B (NP)	1.148	1.156

2400 SERIES 1:44 Proportioning

Specific Gravity Range

Part #	Low	High
0FD2401-3B (NP)	1.208	1.218
0FD2418-3B (NP)	1.205	1.215
0FD2419-3B (NP)	1.209	1.219
0FD2426-3B (NP)	1.210	1.220
0FD2441-3B (NP)	1.206	1.216

3000 SERIES 1:34 Proportioning

Specific Gravity Range

Part #	Low	High
0FD3006-3B (NP)	1.171	1.179
0FD3007-3B (NP)	1.169	1.177
0FD3013-3B (NP)	1.166	1.174
0FD3018-3B (NP)	1.168	1.176
0FD3026-3B (NP)	1.170	1.178
0FD3041-3B (NP)	1.167	1.175

Table 1