



Fresenius Medical Care

SERVICE BULLETIN

Equipment: 2008H Hemodialysis Machines
Bulletin: 96-FRH-022 Rev A
Subject: Concentrate Transfer Instructions

Written By	Reviewed By	Approved By	Effective Date
J. Fox (TSD)	Mark M. Lowery	ECO# 13403	3/31/04

Distribution: Nat'l Svc Mgr (M. Lowery), TS Mgr (K. Cogswell), QS (M. Owen), QE (R. Fox), Eng (D. Gavin), DTS (M. Crnkovich), Tng (B. Hassard)

1.0 PURPOSE

To provide instructions on how to use the Transfer Data feature provided in the 2008H Hemodialysis System.

NOTE: All 2008H machines involved with the transfer must use software version 8.02 or later to ensure correct data transfer.

2.0 PROCEDURE

CAUTION: Do not attempt to transfer concentrate data from a 2008K machine to a 2008H machine nor from a 2008H to a 2008K machine

Machines must be OFF at the onset of this procedure.

- 1) Pull out the monitors of the 2008H machines involved in the data transfer. Reposition the cable connected at P6 of the Functional Board to P9 on both machines.
- 2) Connect cable P/N 150394 to the RS-232 ports of the source machine and destination machine.
- 3) Set the service switch on the back of the 2008H machines to "**SERVICE**" and power the machines ON.
- 4) On the destination machine, select **(6) "TRANSFER DATA"** in the Option Menu and then select "**RECEIVE DATA**".
- 5) On the source machine, select **(6) "TRANSFER DATA"** in the Option Menu and then select "**TRANSMIT CONCS**". When the destination machine displays "**SUCCESS**", the data transfer is complete. If "**SUCCESS**" does not appear on the screen within 20 seconds, check all cable connections and repeat the process.
- 6) Upon completion re-connect the ribbon cable from P9 to P6 on the Functional boards before re-assembly of the monitors.

If there are any questions regarding this bulletin, contact Fresenius Technical Support at 800-227-2572.