



# Fresenius Medical Care SERVICE BULLETIN

Equipment: 2008H  
Bulletin: 98-FRH-004  
Subject: Activation of the Online Pressure Holding Test

Written By	Reviewed By	Approved By	Effective Date
J. Fox	Michael Owen	ECO# 6603	11/3/98

Distribution: Nat'l Svc Mgr (M. Lowery), Tech Support Mgr (K. Cogswell), QSRA (M. Owen), Eng (D.Gavin), Tng (B. Hassard)

## 1.0 PURPOSE

To inform all Field Service and Technical Support personnel of the process used to activate the Online Pressure Holding Test (Online PHT) option on the 2008H Hemodialysis machine.

## 2.0 OVERVIEW

Fresenius recommends the activation of the Online PHT option. Once activated, the Online PHT option performs a pressure holding test every 12 minutes of a dialysis treatment. For a more detailed description of the Online PHT option, refer to the attached document "DESCRIPTION OF THE 2008H ONLINE PHT OPERATION" (P/N 500770).

## 3.0 PROCEDURE

The following steps are used to activate the Online PHT option:

- With the machine **OFF** set the service switch on the back of the machine into the service position.
- Turn the machine **ON** using the power switch on the front panel.
- Once the Service Mode screen appears, use the [↑] and [↓] keys to scroll down to **OPTIONS**. Press the [SET] key to display the **OPTIONS** menu.
- Use the [↑] and [↓] keys again to select **HARDWARE OPTIONS**. Press the [SET] key.
- Once the **HARDWARE OPTIONS** menu appears, use the [↑] and [↓] keys to move the arrow to **ONLINE PHT**.
- Press the [SET] key to toggle the NO next to **ONLINE PHT** to YES.
- Press the [RESET] key twice to return to the Service Mode screen.